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ES

Introduction

Snoring is a widespread problem from which the sleeping partner particularly suffers.

Persistent snoring can lead to sleep apnoea with far-reaching damage to health. The **Friedensstifter**[®] system not only prevents snoring, but it can also be used as a test device prior to a more complex therapy with protrusion splints.

The **Friedensstifter**[®] devices are made from thermo-plastic material and take into account the individual advancement possibilities of the patients. I would like to wish your patients and their sleeping partners a undisturbed restful sleep.

Prof. Dr. Rolf Hinz

The **Friedensstifter**[®] System by Prof. Hinz

The **Friedensstifter**[®] system with prefabricated thermoplastic devices is the first prefabricated treatment system that offers multi-stage diagnosis-relevant treatment devices.

The two prefabricated thermoplastic aligners differ significantly from other prefabricated products with the same objective: Only the **Friedensstifter**[®] system individually takes into account both the scope of different advancement possibilities as well as the different bite positions (Cl. I and Cl. II).

Most people have a (neutral) Cl. I tooth system, approx. 25 – 30 % have a posterior lower jaw position with apparently protruded upper teeth (Cl. II).

The starter set includes four devices with two different advancement fixations of the lower jaw, which correspond to the individual corresponding to the patient's individual advancement. The starter set includes a digi-


tal thermometer, eight wooden mouth spatulas which keeps the air passage open during the adjustment pressure of the heated devices and facilitates handling with the **Friedensstifter**[®] devices in the heated water bath.

Initial position (Angle class)	Maximum lower jaw advancement	Therapeutic advancement	Selection of the Friedensstifter [®] device
CL I	up to 9 mm	approx. 6 mm	Friedensstifter[®] CL I 98081
CL II	up to 9 mm	approx. 6 mm	Friedensstifter[®] CL II 98084

Therapeutic effect of the Friedensstifter®

Due to the preset advancement of the lower jaw, which corresponds to its measured maximum advancement possibility, the tongue and the adjacent soft tissue of the upper airway come forward. This keeps the airway open during sleep and harmless and obstructive snoring is prevented. The effect is however only “symptomatic”, i.e. a cure is not possible, as all sleep therapy devices only work during the night in which they are worn.

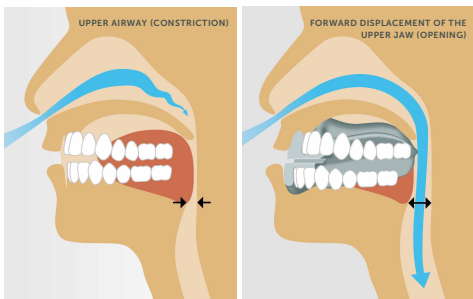Advancement possibilities

The **Friedensstifter**[®] system has an advancement range of 6 mm. This applies to neutral bite cases (Cl. I) as well as distal bite cases (Cl. II) with large anterior tooth distances. The marking of the transparent devices is in the front above and below the air passage.

98081 1 for Cl. I

98084 K2 for Cl. II

Preparation for direct adjustment in the mouth

The prefabricated devices made of thermoplastic material have to be heated in a hot water bath at 70° C for for approx. 30 seconds. Provide a kettle, a thermometer and a high bowl for the water bath.



Heat the water to approx. 80° C, pour it into a bowl and check the water temperature with the thermometer. **When the water has cooled down to 70° C**, put the "Friedensstifter[®]" into the water bath. It must be filled

with water completely in order to ensure even heating. **After 30 seconds, the treatment device is sufficiently plastic** to be adjusted directly in the mouth.

The lower part of the jaw is adjusted first. In order to facilitate the upper part of the jaw, the patient has to move **the the lower jaw forward and bite firmly at the same time.**

The whole Friedensstifter® device has to be in the water.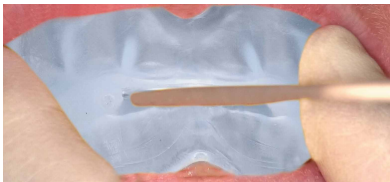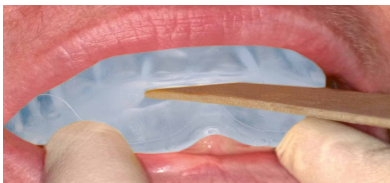

Direct adjustment of the Friedensstifter® device in the mouth

After 30 seconds, remove the device from the mouth – it is ready for use.

Something that you should still pay attention to:

1. **Before heating, insert the wooden spatula into the breathing** passage and leave it there during the adjustment in order to prevent it from closing.
2. The device is cleaned under cold **running water**.
3. **Roughness on the surface of the device**, which only occurs when it is overheated, can easily be polished smooth.

IST® devices

The intraoral snoring therapy devices (IST®) based on the work of Prof. Hinz are high-quality protrusion splints with a high level of comfort for the wearer. They have a delicate design and offer a high level of freedom of movement in the lower jaw. They are individually made by the dental technician

For example:
IST®classic ^{New} with
two lateral adjustable
guide bars.

Further questions or problems?

The Dr. Hinz dental team will be happy to advise you:

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